English

Addendum for the Mobile Care Web Viewer User's Reference Manual

5822255-99

2nd edition

2022-02-10

© 2019, 2022 General Electric Company. All rights reserved.

All specifications are subject to change without notice.

GE Healthcare Finland Oy

Kuortaneenkatu 2

FI-00510 Helsinki, Finland

Tel: +358 10 39411

www.gehealthcare.com

Intended use of this addendum

This addendum covers necessary updates to the Mobile Care Web Viewer User's Reference Manual.

Macintosh operating system compatibility change

The Web Viewer client is no longer compatible with Macintosh desktop, laptop, or tablet computers.

Changes to the How to start and exit the Web Viewer chapter

The procedure for starting the Web Viewer has changed. To start the application:

1. Click the Web Viewer icon on your desktop. If the Web Viewer icon is not visible, consult the Mobile Care Web Viewer Technical Manual or contact your system administrator.
2. When the login screen appears, check that the server IP address is visible. If it is not visible, contact your system administrator to obtain the IP address and enter the information in the corresponding text box.
3. Give your user name and password. For changing your password, see the user's reference manual.
4. Confirm and continue as instructed in the user's reference manual.

Using the Web Viewer for the first time

The instructions in this section are no longer valid. For instructions related to installation, consult the Mobile Care Web Viewer Technical Manual or contact your system administrator.

Creating a shortcut for the Web Viewer

The instructions in this section are no longer valid.

Configuration pages

The Pocket Viewer and Cellular Viewer are no longer available.

Changes to the General specifications chapter

The following sections replace the General specifications chapter of the user's reference manual.

Technical specifications

For the computer and display technical specifications, refer to the user documentation delivered with the respective equipment.

Web Viewer client application performance

Maximum delay for waveforms, parameters and alarms is 10 seconds under normal circumstances.

Web Viewer client computer requirements

The Web Viewer client computers must meet the following minimum requirements:

- Windows desktop, laptop, or tablet computer:
 - Windows 10
 - Citrix Presentation server version 6.5, Citrix XenApp version 6.5 for Windows Server 2008 R2, Citrix XenApp version 7.13 for Windows Server 2016, or Citrix XenApp version 7.15 for Windows Server 2016
 - EDGE 96.0.1054.62, Firefox 95.0, Firefox ESR 91.4.1, or Chrome 96.0.4664.110
 - JRE 8
 - 1024 x 768 resolution or greater
 - 256 colors or greater
- Network connection:
 - LAN connection with TCP/IP networking installed.

Always check the Web Viewer client functionality after installing or updating the operating system, browser, or JRE. For more information, see the Mobile Care Web Viewer Technical Manual.

Changes to the Device compatibility chapter

The following sections replace the Device compatibility section of the user's reference manual.

Device compatibility

- | | |
|------|--|
| NOTE | Since new product versions are introduced due to continuous product development, it is recommended that you contact your local GE representative for the latest updated lists. |
| NOTE | The system devices connected to Datex-Ohmeda S/5 network are no longer supported as iCentral service life has ended in January 2020. |

Compatible MC Network devices

Device	Supported versions/devices
ApexPro Telemetry System	4.3, 5.0
Dash 3000/4000/5000 Patient Monitor	7.2, 7.3
Solar 8000i Patient Monitor	4E, 4F, 5.3, 5.7
CARESCAPE Monitor B850	v2
CARESCAPE Monitor B650	v2
CARESCAPE Monitor B450	v2
CARESCAPE B850	v3
CARESCAPE B650	v3
CARESCAPE B450	v3
B40i Patient Monitor	v2, v2.1, v3
CARESCAPE Central Station	v2, v3
B105 Patient Monitor	v1, v2, v3
B125 Patient Monitor	v1, v2, v3
B155 Patient Monitor	v3

Other compatible devices

Device	Supported versions/devices
Unity Network Interface Device	<p>1A, 3B, 4A, 4B, 5A, 5B, 5C, 5D, 6A, 6B, 6C, 7A, 8A, 9A, 10A, including the following devices:</p> <ul style="list-style-type: none"> • All supported anesthesia machines • All supported continuous cardiac output monitors, including the Novametrics NICO 7300 Cardiac Output Monitor • All supported gas analyzers • All supported pulse oximeters • All supported ventilators <p>For more information, see the documentation accompanying the device.</p>

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